14302 Vitro Fil - Instr de Uso PEI

165mm

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PROFESSIONAL USE ONLY.

Composition:

VITRO FIL Powder contains: Fluorine Strontium Aluminium Silicate, Dehydrated Polyacrylic Acid and Iron Oxide.

VITRO FIL Liquid contains: Polyacrylic Acid, Tartaric Acid and Distilled Water

VITRO CONDICIONADOR contains: Polyacrylic Acid, Glycerin, Aerosil 200, Methylene Blue CI 52015 and Deionized Water.

VITRO FIL is a chemically activated glass ionomer cement of good esthetic properties; it is acid/basic, radiopaque due to the strontium action and is easily manipulated. It presents the features that are common to other glass ionomers, i. e., biocompatibility, high chemical adhesion, low solubility, adhesion to wet surfaces and continuous fluoride release.

Indications:

VITRO FIL is recommended in Class III or V restorations, as an intermediary material in Class I and II dental cavities under high stress, in constructions of filling cores, small Class I restorations, restorations of deciduous teeth and as sealant for fissures.

Presentations:

Box containing: 10g bottle of **VITRO FIL** powder + 8mL bottle of **VITRO FIL** liquid + 10mL bottle of **VITRO CONDICIONADOR** + 1 measuring spoon + 1 mixing pad for spatulation. Powder available in the following colors: A1, A2, A3, A3.5, B2, C2, C3 or DGO.

Box containing: 10g bottle of **VITRO FIL** powder + 8mL bottle of **VITRO FIL** liquid + 1 measuring spoon + 1 mixing pad for spatulation. Powder available in the following colors: A1, A2, A3, A3.5, B2, C2, C3 or DGO.

Box containing: 5g bottle of **VITRO FIL** powder + 4mL bottle of **VITRO FIL** liquid + 1 measuring spoon. Powder available in the following colors: A1, A2, A3, A3.5, B2, C2, C3 or DGO.

Box containing: 10g bottle of **VITRO FIL** powder + 1 measuring spoon. Powder available in the following colors: A1, A2, A3, A3.5, B2, C2, C3 or DGO.

Box containing: 8mL bottle of **VITRO FIL** liquid.

Box containing: 10mL bottle of **VITRO CONDICIONADOR**.

Storage and Conservation:

VITRO FIL must be stored between 15°C and 30°C. Do not expose to excessive heat. Keep the containers of powder and liquid tightly closed after use. Under these conditions, the product is valid for up to 2 years as of the manufacturing date. Do not use this product if shelf life had expired.

Instructions for Use:

The use of gloves is recommended during the procedure.

Attention: The patient must be informed on the risk to occur sensitivity in the place of the procedure. This sensitization can be minimized through the use of proper isolation (rubber sheet) and pulp protection.

Do not apply **VITRO FIL** on the pulp, or over a dentin layer less than 0.5 mm thick. In such cases, if endodontic treatment is not necessary, a calcium hydroxide cement should be previously applied instead.

Preparation

Preparation is limited to the cavity left after removal of the decayed tissue and/or old restoration.

Tooth Conditioning

Before applying **VITRO FIL**, you should apply **VITRO CONDICIONADOR** over the cavity surface (enamel and dentin) for 10 seconds. After that, rinse it thoroughly with water and carefully dry it with air or sterilized cotton pellets. Keep the surface wet.

Note: Glass ionomers, unlike other materials, do not work well on a dehydrated surface.

Dosage

Gently shake the bottle, to release the powder. Fill the measuring spoon with powder and level it in the stopper. The proportion used should be of 1 measuring spoon of the powder to 1 drop of the liquid. The liquid should be dispensed while holding the bottle in a vertical position. Any crystals present in the neck should be removed, to assure that there are not air bubbles in the dispensed drop. Keep the powder and liquid bottles tightly closed after use, avoiding exposure to light.

Manipulation

VITRO FIL must be manipulated at a temperature between 20-30°C. With a metal or plastic spatula, prepare the mixture over an appropriate mixing pad or glass slab. The ideal procedure is to add the powder into the liquid in 2 portions. Mix the powder with the liquid until a homogenous consistence is obtained. The resulting mixture should be shiny and free of granules.

VITRO FIL with tip and syringe.

Application

Apply **VITRO FIL** with glass ionomer injection tip and syringe. During application, do not remove the tip from the material already applied in order to prevent the formation of air bubbles. The appropriate technique for each kind of indication must be followed. Whenever possible, apply some pressure using an acetate matrix and your finger over the applied material until complete setting is obtained. Then, remove the matrix and immediately apply a varnish on top of the restoration.

Finishing and Polishing

Wait 6 minutes before you start the finishing procedure. Depending on the area, excess material can be removed with finishing diamond burs for composite resins. Use low rotation. Polishing can be performed with silicone tips in low rotation. Apply a new varnish layer and instruct the patient not to chew for about one hour.

Properties of Use

Times given below are valid for the specific temperature and humidity conditions described. Higher temperatures will **reduce** and lower temperatures will **increase** working time.

Mixing Time (21-23°C, 50-52% HR)	30 sec
Working Time (21-23°C, 50-52% HR)	> 2 min 10 sec
Net Setting Time (36-38°C, > 90% HR)	2 min 4 min

The product tends to adhere to metal instruments; for this reason, it is recommended that these instruments be cleaned with cold water before drying of the residues.

Product wastes must not be reused. They must be disposed as the local law in force.

Precautions:

The liquid and the prepared cement can be corrosive. Do not allow contact with soft tissues or the skin. In case of contact, you should immediately remove the material with hydrophilic cotton soaked with water and rinse thoroughly with water. Avoid contact with the eyes; in case of accidental contact, wash the eyes immediately with water and see a doctor. Do not mix the powder or liquid which are components of this product with any other glass ionomer-based product. This product must be used only by surgeon dentists and for the intended use described in the Instructions for Use. Avoid contact with eugenol-based products, since they can delay the action of glass ionomers.

Contraindications:

Avoid the use in patients with previous history of allergy to glass ionomer cements. In case of allergic reaction, discontinue the use immediately and advise the patient to see a doctor.

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DFL Indústria e Comércio S.A.