ELITE MODEL

THIXOTROPIC

GESSO TIPO 3 PER MODELLI IN GENERE E ANTAGONISTI
TYPE 3 DENTAL STONE FOR ANTAGONISTS AND MODELS IN GENERAL
HARTGIPS, KLASSE 3, FÜR SITUATIONS- UND GEGENBISSMODELLE
PLÂTRE TYPE 3 POUR MODÈLES EN GÉNÉRAL ET D'ANTAGONISTES
YESO TIPO 3 PARA MODELOS EN GENERAL Y ANTAGONISTAS

WATER / POWDER RATIO	30 ml / 100 g
WORKING TIME	7'
SETTING TIME (VICAT)*	12'
SETTING EXPANSION 2h*	0.04 %
COMPRESSIVE STRENGTH 1h*	31 MPa

ELITE MODEL FAST

THIXOTROPIC

GESSO RAPIDO TIPO 3 PER MODELLI IN GENERE E ANTAGONISTI
TYPE 3 FAST DENTAL STONE FOR ANTAGONISTS AND MODELS IN GENERAL
SCHNELLABBINDENDER HARTGIPS, KLASSE 3, FÜR SITUATIONS- UND GEGENBISSMODELLE
PLÂTRE RAPIDE TYPE 3 POUR MODÈLES EN GÉNÉRAL ET D'ANTAGONISTES
YESO RÁPIDO TIPO 3 PARA MODELOS EN GENERAL Y ANTAGONISTAS

WATER / POWDER RATIO	30 ml / 100 g
WORKING TIME	4'
SETTING TIME (VICAT)*	8'
SETTING EXPANSION 2h*	0.1 %
COMPRESSIVE STRENGTH 1h*	36 MPa

ELITE ARTI

THIXOTROPIC

GESSO TIPO 3 PER ARTICOLATORI
TYPE 3 ARTICULATING DENTAL STONE
ARTIKULATIONSGIPS, KLASSE 3
PLÂTRE TYPE 3 POUR ARTICULATEURS
YESO TIPO 3 PARA ARTICULADORES

WATER / POWDER RATIO	30 ml / 100 g
WORKING TIME	3'
SETTING TIME (VICAT)*	5'
SETTING EXPANSION 2h*	0.02 %
COMPRESSIVE STRENGTH 1h*	26 MPa

ELITE ARTI FAST

THIXOTROPIC

GESSO RAPIDO TIPO 3 PER ARTICOLATORI
TYPE 3 FAST SETTING ARTICULATING DENTAL STONE
SCHNELLABBINDENDER ARTIKULATIONSGIPS, KLASSE 3
PLÂTRE RAPIDE TYPE 3 POUR ARTICULATEURS
YESO RÁPIDO TIPO 3 PARA ARTICULADORES

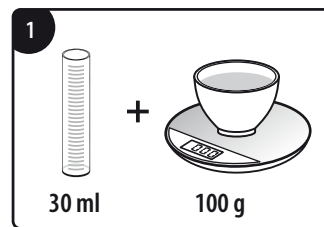
WATER / POWDER RATIO	30 ml / 100 g
WORKING TIME	2'
SETTING TIME (VICAT)*	3'
SETTING EXPANSION 2h*	0.02 %
COMPRESSIVE STRENGTH 1h*	26 MPa

ELITE ORTHO



THIXOTROPIC

GESSO TIPO 3 PER MODELLI IN ORTODONZIA
TYPE 3 DENTAL STONE FOR ORTHODONTIC MODELS
KFO GIPS, KLASSE 3
PLÂTRE TYPE 3 POUR MODÈLES EN ORTHODONTIE
YESO TIPO 3 PARA MODELOS, EN ORTODONCIA



WATER / POWDER RATIO	24 ml / 100 g
WORKING TIME	7'
SETTING TIME (VICAT)*	14'
SETTING EXPANSION 2h*	0.04 %
COMPRESSIVE STRENGTH 1h*	32 MPa

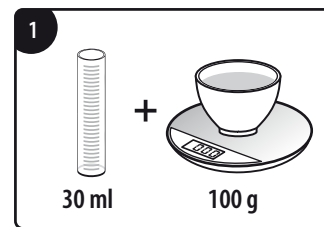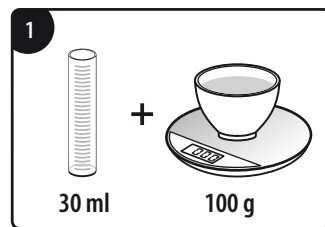
Rapporto acqua/polvere
Water/powder ratio
Wasser/Pulver Verhältnis
Ratio eau/poudre
Dosificación agua/polvo

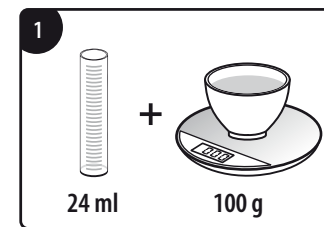
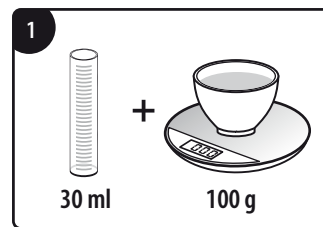
Rapporto acqua/polvere
Water/powder ratio
Wasser/Pulver Verhältnis
Ratio eau/poudre
Dosificación agua/polvo

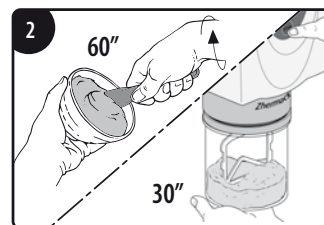
Rapporto acqua/polvere
Water/powder ratio
Wasser/Pulver Verhältnis
Ratio eau/poudre
Dosificación agua/polvo

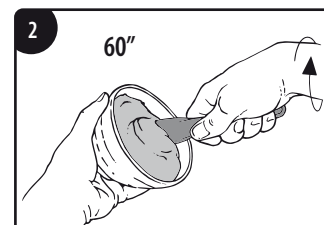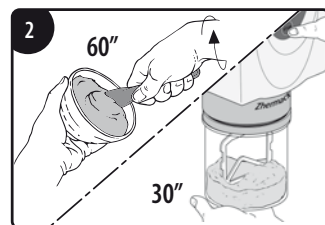
Miscelare per 60/30 secondi
Mix for 60/30 seconds
60/30 Sekunden anmischen
Mélanger pendant 60/30 secondes
Mezclar entro 60/30 segundos

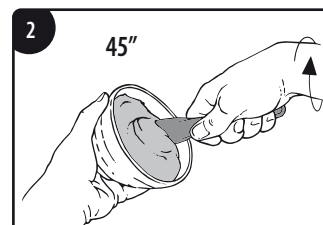
Miscelare per 60 secondi
Mix for 60 seconds
60 Sekunden anmischen
Mélanger pendant 60 secondes
Mezclar entro 60 segundos


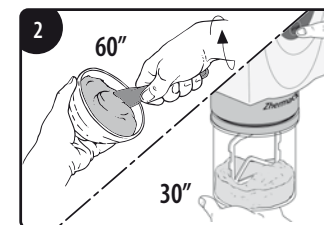
Miscelare per 45 secondi
Mix for 45 seconds
45 Sekunden anmischen
Mélanger pendant 45 secondes
Mezclar entro 45 segundos


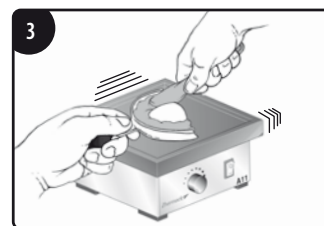
Miscelare per 60/30 secondi
Mix for 60/30 seconds
60/30 Sekunden anmischen
Mélanger pendant 60/30 secondes
Mezclar entro 60/30 segundos

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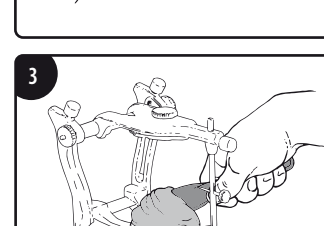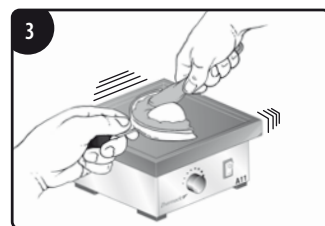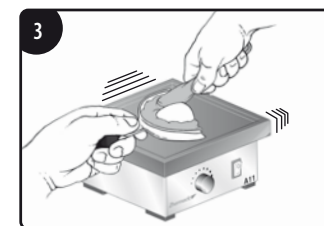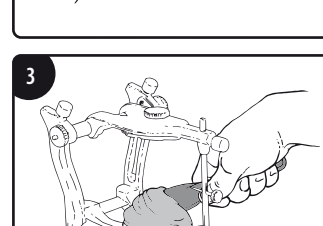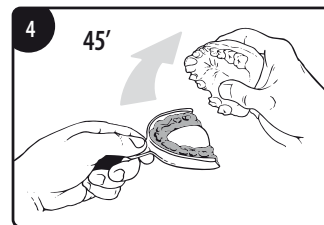
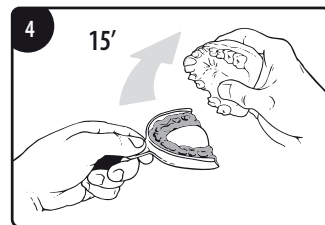
EN - WARNING: Be sure to properly disinfect the impression before pouring. PRECAUTIONS: Be aware that during hardening the mixed material may develop enough heat to cause dermal burns. Do not touch the material during hardening. Ensure adequate ventilation. Avoid dust formation and use respiratory and eye protection. Wear gloves to prevent prolonged or repeated skin contact. The times indicated may vary according to room temperature and mixing technique. Dental stone. This product is humidity-sensitive, store in a dry place.

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


WATER / POWDER RATIO	21 ml / 100 g
WORKING TIME	12'
SETTING TIME (VICAT)*	14'
SETTING EXPANSION 2h*	0.08 %
COMPRESSIVE STRENGTH 1h*	50 MPa

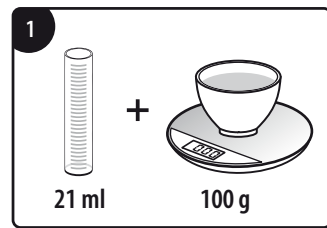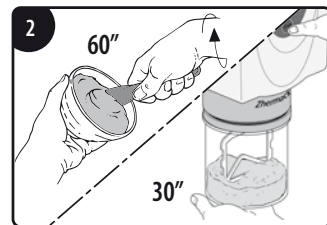
Rapporto acqua/polvere
 Water/powder ratio
 Wasser/Pulver Verhältnis
 Ratio eau/poudre
 Dosificación agua/polvo

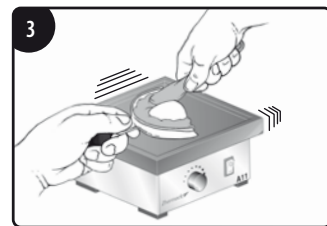Miscelare per 60/30 secondi
 Mix for 60/30 seconds
 60/30 Sekunden anmischen
 Mélanger pendant 60/30 secondes
 Mezclar entro 60/30 segundos

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

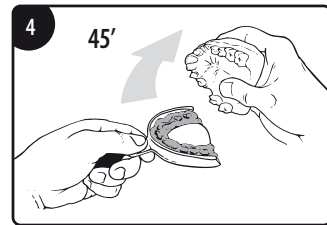
WATER / POWDER RATIO	20 ml / 100 g
WORKING TIME	12'
SETTING TIME (VICAT)*	14'
SETTING EXPANSION 2h*	0.08 %
COMPRESSIVE STRENGTH 1h*	52 MPa

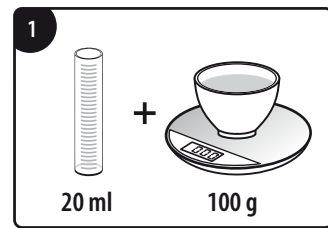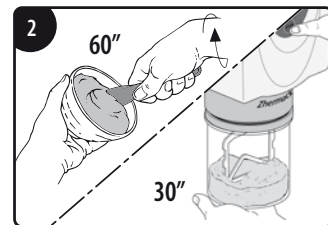
Rapporto acqua/polvere
 Water/powder ratio
 Wasser/Pulver Verhältnis
 Ratio eau/poudre
 Dosificación agua/polvo

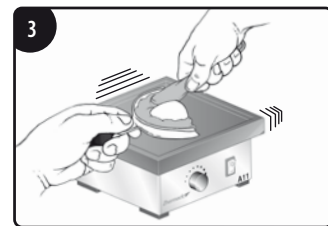Miscelare per 60/30 secondi
 Mix for 60/30 seconds
 60/30 Sekunden anmischen
 Mélanger pendant 60/30 secondes
 Mezclar entro 60/30 segundos

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

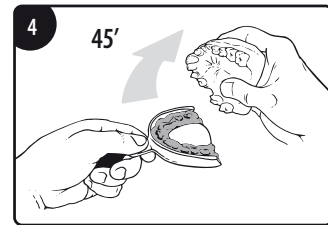
WATER / POWDER RATIO	20 ml / 100 g
WORKING TIME	5'
SETTING TIME (VICAT)*	9'
SETTING EXPANSION 2h*	0.08 %
COMPRESSIVE STRENGTH 1h*	52 MPa

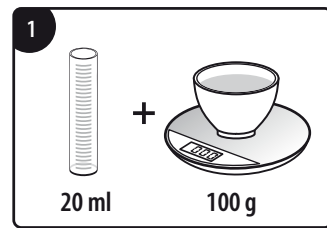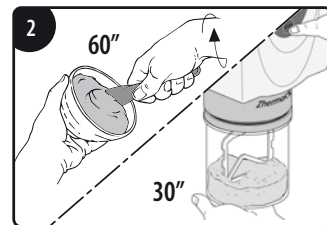
Rapporto acqua/polvere
 Water/powder ratio
 Wasser/Pulver Verhältnis
 Ratio eau/poudre
 Dosificación agua/polvo

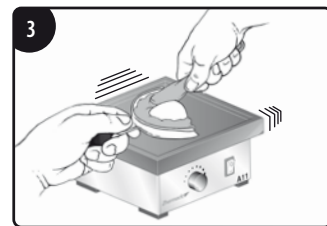Miscelare per 60/30 secondi
 Mix for 60/30 seconds
 60/30 Sekunden anmischen
 Mélanger pendant 60/30 secondes
 Mezclar entro 60/30 segundos

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

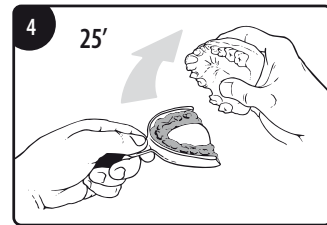
WATER / POWDER RATIO	25 ml / 100 g
WORKING TIME	8'
SETTING TIME (VICAT)*	14'
SETTING EXPANSION 2h*	0.08 %
COMPRESSIVE STRENGTH 1h*	42 MPa

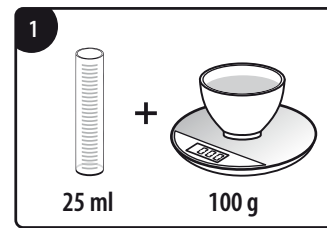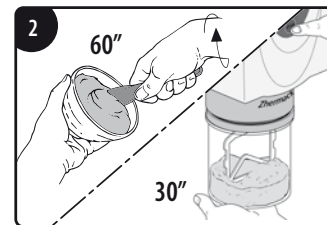
Rapporto acqua/polvere
 Water/powder ratio
 Wasser/Pulver Verhältnis
 Ratio eau/poudre
 Dosificación agua/polvo

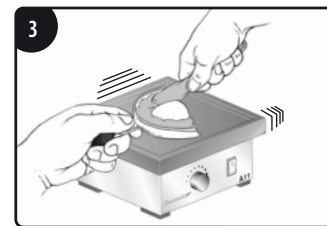Miscelare per 60/30 secondi
 Mix for 60/30 seconds
 60/30 Sekunden anmischen
 Mélanger pendant 60/30 secondes
 Mezclar entro 60/30 segundos

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

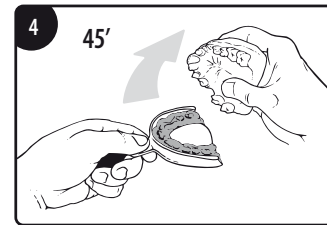
WATER / POWDER RATIO	25 ml / 100 g
WORKING TIME	3' 30"
SETTING TIME (VICAT)*	6'
SETTING EXPANSION 2h*	0.09 %
COMPRESSIVE STRENGTH 1h*	42 MPa

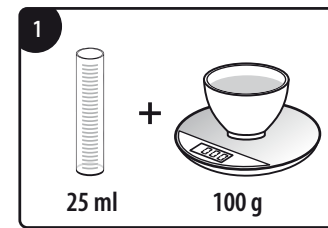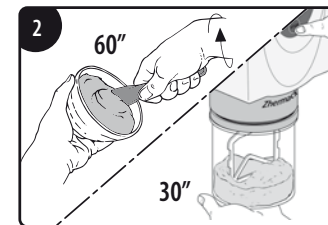
Rapporto acqua/polvere
 Water/powder ratio
 Wasser/Pulver Verhältnis
 Ratio eau/poudre
 Dosificación agua/polvo

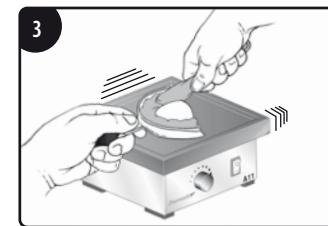Miscelare per 60/30 secondi
 Mix for 60/30 seconds
 60/30 Sekunden anmischen
 Mélanger pendant 60/30 secondes
 Mezclar entro 60/30 segundos

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

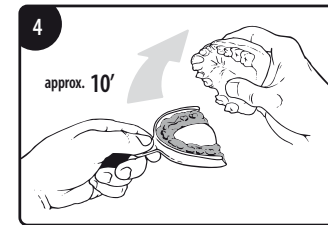
WATER / POWDER RATIO	23 ml / 100 g
WORKING TIME	5'
SETTING TIME (VICAT)*	18'
SETTING EXPANSION 2h*	0.05 %
COMPRESSIVE STRENGTH 1h*	40 MPa

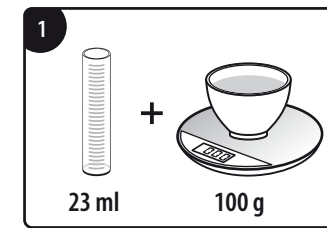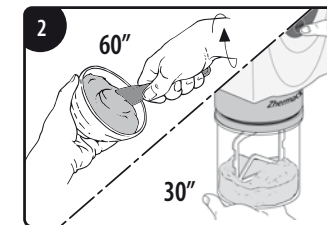Rapporto acqua/polvere
 Water/powder ratio
 Wasser/Pulver Verhältnis
 Ratio eau/poudre
 Dosificación agua/polvo

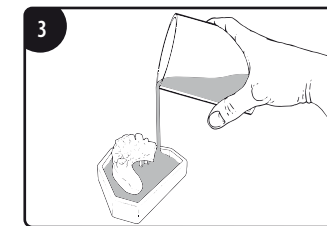Miscelare per 60/30 secondi
 Mix for 60/30 seconds
 60/30 Sekunden anmischen
 Mélanger pendant 60/30 secondes
 Mezclar entro 60/30 segundos

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